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02/05/22
Clément EDEL

This instructions for use contains important information.
Please read carefully and retain for future reference.

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EN SPRAY & CLEAN

INDICATIONS

Spray & Clean is a preservative free solution indicated for cleaning of all rigid gas permeable contact lenses.
The specific device ensures sterility of the solution during use.

DIRECTIONS

Before each contact lens manipulation, wash your hands.
Remove respectively your lenses from the eye into the palm of your hand.
Spray one shot of solution on each side of the lens.
Rub the lens carefully in your palm with your fingers or between fingers.
Put the lens on related lens holder of the lens case and rinse the lens thoroughly with a rinsing solution.
Fill your lens case with a rinsing solution and screw the cap. Before wearing lenses always rinse the lenses with an adapted rinsing solution. Throw away the remaining solution from lens case.
Frequency of use: can be used every day. Follow the instructions given by your contact lens specialist.

Before wearing lenses:

Rinse the lenses with an adapted rinsing solution. The lens can be worn. Throw away the remaining storage solution from lens case.

WARNINGS

Keep out of children's reach.
Consult your doctor or eye care professional before using any eye medication with contact lenses.
It is essential to rinse the lenses with an adapted rinsing solution after use of Spray & Clean. In case of misuse, take out lenses, rinse immediately your eyes thoroughly with water and consult your eye care practitioner.

PRECAUTIONS

- Validity after initial use : 3 months.
- Do not touch dispenser tip. 
- Store at room temperature. 
- Observe the expiry date printed on the label.
- Important : do not swallow or drink.
- Do not spray in the nose.
- Do not put Spray & Clean directly into your eye. 
- Use your regular solution for rinsing. 
- Store out of direct sunlight. 
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

CONTRA-INDICATIONS

- Do not use for soft lenses.
- Do not use in case of allergic background to one of ingredients. In case of problems contact your eye care practitioner.

CONTENTS

Sodium Olefine sulfonate.....	0,40 %
Polyoxyethylene Alkyl Ether.....	0,60 %
EDTA.....	0,57 %
Purified water.....	q.s. 100

PACKAGING

Spray & Clean is available in 15 ml bottle.

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© 2009,